


Quick Installation Guide V16112017



Installation materials:

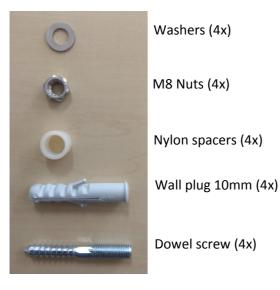


A complete manual is available at www.sixcase.com

Suitable for installation on brick/concrete or wood

IMPORTANT:

- Minimum required wall thickness: 3mm
- Install on a flat surface
- Always use the spacers
- Recommended height: 1.5 1.6m upper holes (above ground)
- Install in shadow
- Place adapter indoors

Installation

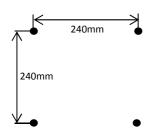

1: Drill 10mm holes (4x)



2: Insert Plugs and dowel screws (20mm protrusion)




3: Place spacers

- 4: Feeder cable: Drill a 12 mm hole through the wall
- d dowel screws (20mm pr
 - 5: place washers and M8 Nuts on the inside






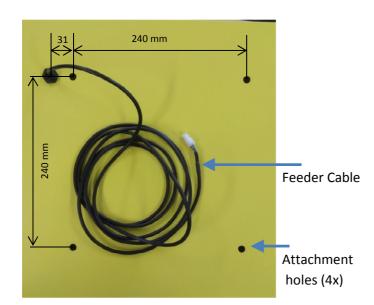





For the SC1240 only: code lock code change

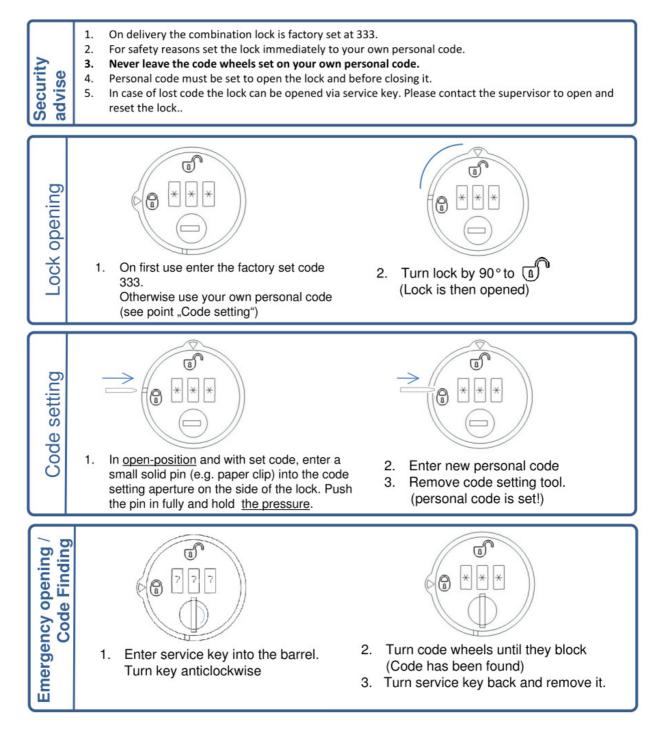

Always verify the code change by opening the SmartCase AED cabinet using the new code